



SCAN - SCRATCH - SMELL - SELECT - PASS

Rapid & Objective COVID-19 Symptom Screening Test




info@blvdmedical.com



Corowell Benefits

- FDA class 1 and 2 medical device
- CE approved
- Pre-screen large groups of people
- Highly accurate
- User friendly experience and interface.
- Results are sent to your App as an encrypted digital pass that can be scanned and validated by using the Corowell Guard App at events, schools, airports, etc.
- Process takes less than two minutes.

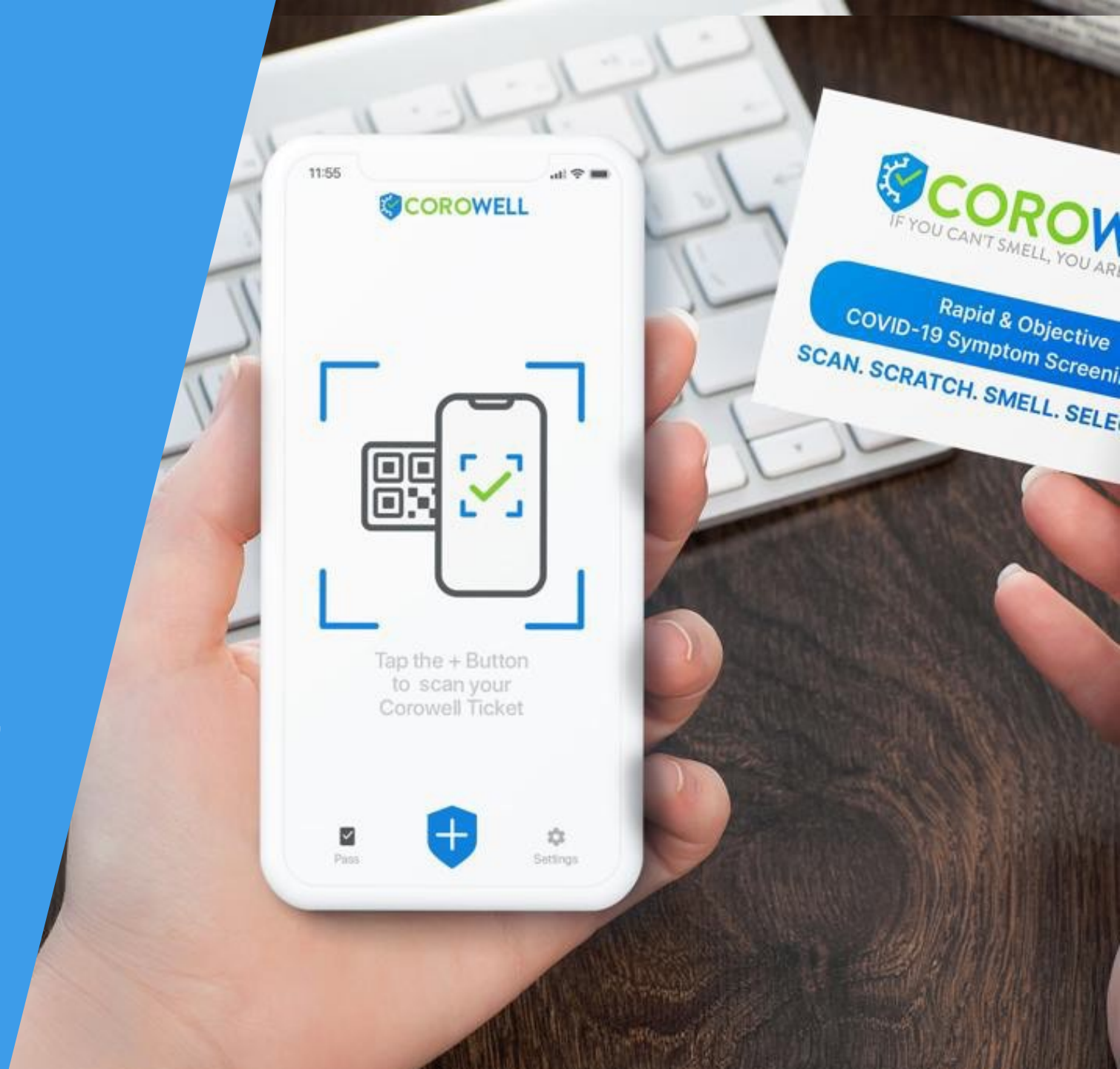

Corowell Symptom Screening Test is simple and easy, takes less than two-minutes



Scan Card



Scratch and Smell
the scented area



Select the correct
sent in the app



Complete the
COVID-19 survey






Get a **PASS** with
name and issue
date or **FAIL** the
test

The Corowell app

This is how it works:

1. Scan the QR Code inside the Corowell Ticket (sold separately)
2. Scratch the Scent Field
3. Smell it
4. Select the correct Scent in the app
5. Answer the Questionnaire
6. Get your Pass

Add the Corowell Pass to your Apple Wallet and present it quickly.

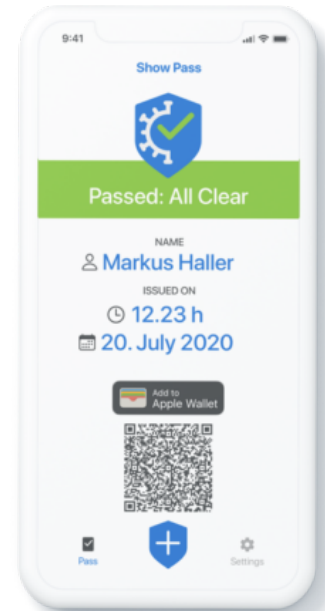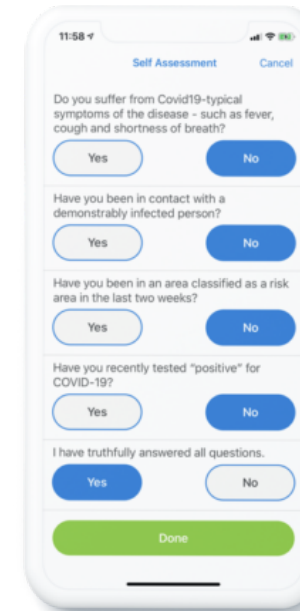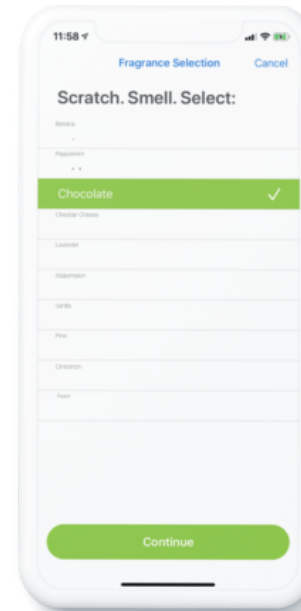

Ratings and Reviews

5.0 out of 5

5 Ratings



Ashol K Patel, 08/10/2021

Easy to Use

This test was very easy & I didn't have to have a swab jammed up my nose. I got the results in less than 60 seconds I use these at my hotels they work great!



Mick Dogg, 08/05/2021

Great Product

This product works great with the app I got my results in 1 min or so. Perfect for screening COVID symptoms



None 9876, 08/04/2021

Interesting and seems to work?

First - very easy to use. Get the app, scan the QR code, do the sniff test, answer some questions get a result within moments.

Secondly, so much better than swabbing.....

The Corowell Ticket

Available in Retail and Bulk Wholesale Packaging

Packaging		NHRIC/Item Code
Corowell 3	Tickets pack	72678-004-02
Corowell 5	Tickets pack	72678-004-03
Corowell 10	Tickets pack	72678-004-04
Corowell 125	Tickets pack	72678-004-05



Clinical

Corowell Rapid Screening COVID Smell Test

Corowell is a screening test. To determine if someone is clear of COVID. This is not a replacement for PCR, but a first means of defense against the proliferation of the disease. If a person passes the Corowell test there is 98% accuracy that the person does not have COVID.

This is the same accuracy as the PCR. Detection of the virus is also within an hour of infection with the Corowell Test, same as the PCR, but much faster than Antigen. Corowell Rapid Screening COVID Smell Test is a tool to be used to keep unaffected individuals from mingling with those infected with COVID, keeping workplaces, institutions, entertainment facilities, and public gatherings free of COVID exposure.

Clinical studies available upon request



	Corowell	PCR Test	Fever Test	Antibody Test	Antigen Tests
Specificity & Sensitivity	✓	✓	✗	✓	✓
Immediate results	✓	✗	✓	✗	✓/✗
Suitable for mass testing	✓	✗	✓	✗	✓/✗
No personal contact required	✓	✗	✗	✗	✓
Earliest possible prediction	Day 1-3	Day 1-3	Day 5-6	Day 10-17	Day 3-10
Electronic access pass	✓	✗	✗	✗	✓

FDA Establishment Registration & Device Listing

<https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfrl/rl.cfm?lid=711685&lpcd=NRK>

DailyMed Label: Corowell COVID-19 Symptom Screening Test

<https://dailymed.nlm.nih.gov/dailymed/drugInfo.cfm?setid=01759ee7-be7c-4763-ba86-36c7f2302cd7>

Clinical Information

1. Validation Trial – Nov 2020

- ✓ 50 subjects tested on 8 consecutive days (374 tests)
- ✓ 7/50 subjects filtered as "infection-suspects"
- ✓ 3/7 subjects PCR-Positive
- ✓ **ZERO** subjects False Negative

3. Validation Trial – Mar 2020

- ✓ 265 visitors tested at a Concert
- ✓ ALL were tested RT-PCR, Antigen and Corowell
- ✓ 1/265 subjects filtered as "infection-suspects"
- ✓ **One** False Positive
- ✓ ZERO subjects PCR-Positive
- ✓ **ZERO** False Negatives


2. Validation Trial – Jan/Feb 2021

- ✓ 150 subjects test during clinical admission
- ✓ ALL were tested RT-PCR and Corowell
- ✓ 16/150 subjects filtered as "infection-suspects"
- ✓ 6/16 subjects PCR-Positive
- ✓ 5/6 subject detected by Corowell,
- ✓ **One** False Negative

Pooled Validation Trial Data

COROWELL	RT-PCR positive	RT-PCR negative	Total
"suspected of infection"	8	24	32
"NOT suspected of infection"	1	441	442
Total	9	465	474
Sensitivity (pooled)	88.89%		
Specificity (pooled)	94.84%		

WATCH IT WORK

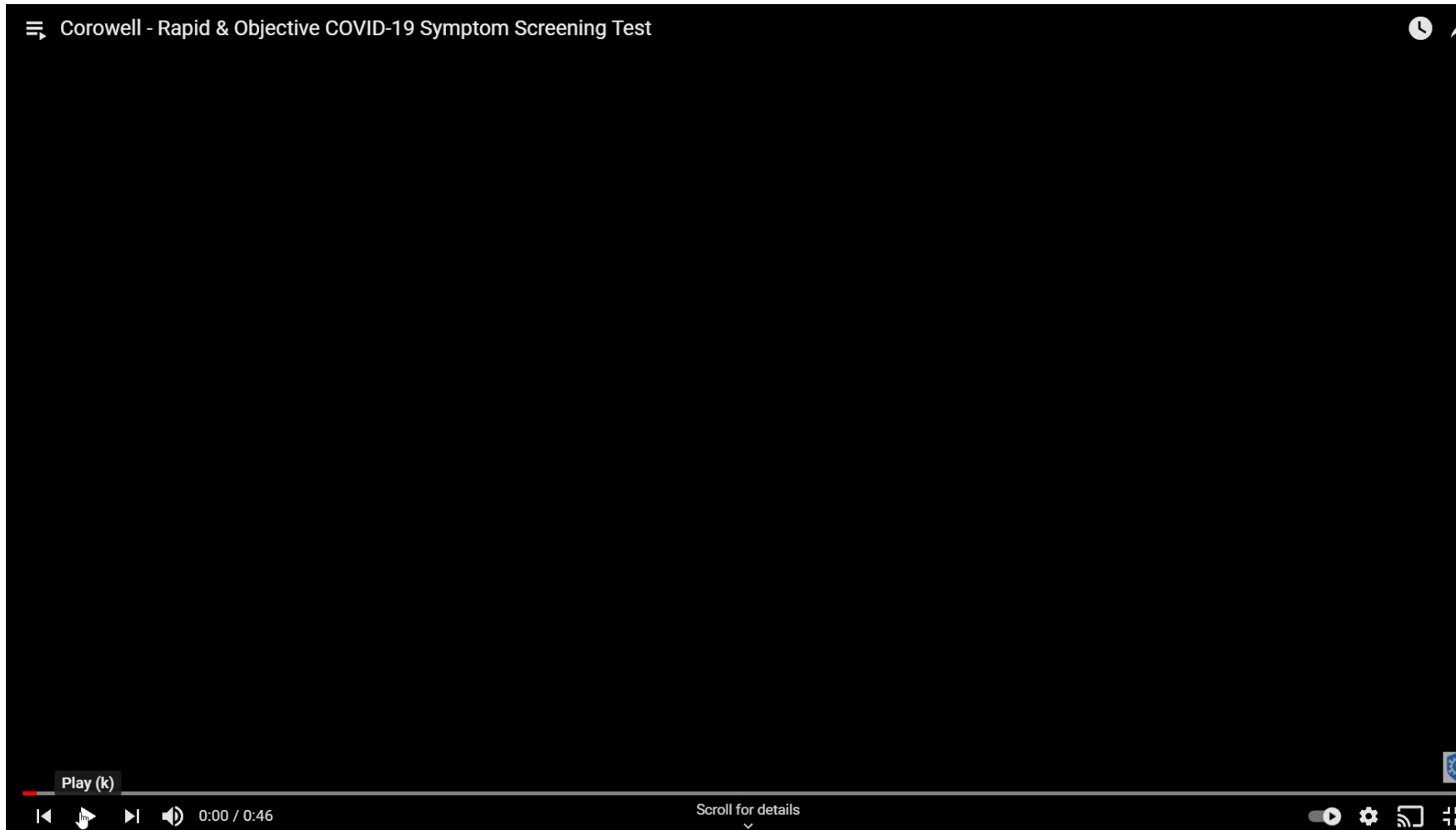



info@blvdmedical.com