


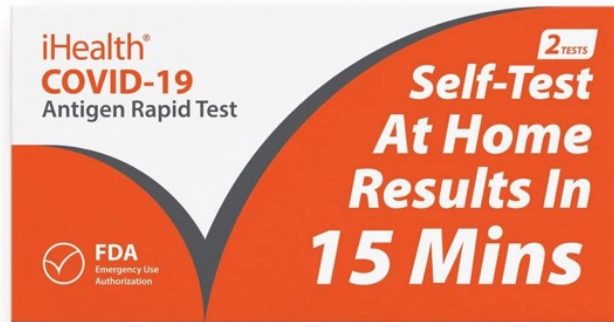

4 Steps. 15 Minutes. Zero Discomfort.

iHealth COVID-19 Antigen Rapid Test is the simplest way to detect SARS-CoV-2 nucleocapsid protein antigen. It only takes 4 steps and 15 minutes to complete the test. The test is also non-invasive. You won't need to collect a sample from deep in your nasal cavity to get accurate results.



Key Highlights



FDA AUTHORIZED 15 MINS SELF-TEST

The test is a 15-minute self-test to detect whether or not an individual has the COVID-19. The test can be completed in the comfort of your own home without the need to ship your sample to a lab.



EASY TO USE ZERO DISCOMFORT

Test can be done by inserting only 1/2 to 3/4 inch with a simple non-invasive nasal swab, easy to use and zero discomfort. Step-by-step instructional videos are available in our app for easy following. (Installation of app is optional.)



FOR AGES 2 AND ABOVE

The self-administered test is recommended for individuals aged 15 years and older. Adult-collection is required for testing children 2-14 years old.



DETECT CURRENT AND NEW COVID VARIANTS

iHealth has completed testing on several heat inactivated variant strains and the iHealth COVID-19 Antigen Rapid Test was able to detect the mutations.



GROUP TESTING MANAGING VIA MOBILE APP

The iHealth Test app allows the administrator of a small group to monitor and track the group members' test results when needed at school, work or event.



An in-home test designed to get accurate results.

Not only is the iHealth COVID-19 Antigen Rapid Test easy and non-invasive, it's a small engineering beauty. The test has a highly-absorbent swab that is individually sealed to help prevent contamination so you can view the results of your in-home test with confidence.



Highly-Absorbent swab



Double-Cap Security Design



Easy-to-Understand Results

Flu or COVID, 15 mins to get peace of mind.

In case of physical discomfort such as fever, sneezing, cough or fatigue, you can quickly detect SARS-CoV-2 nucleocapsid protein antigen by using iHealth COVID-19 Antigen Rapid Test.



Swab

Brush against the inner wall of both nostrils 5 times each in a circular motion with a non-invasive nasal swab. Our nasal swab is soft, highly absorbent, and only needs to be inserted 1/2 - 3/4 inches so you can test yourself comfortably.



Dip

Insert the swab with the sample into the bottom of the tube and stir it in the fluid 15 times. Squeeze the sides of the tube around the swab as you pull it out.



Drip

Put 3 drops of the mixed solution onto the sample port of the COVID-19 test card.







Wait 15 minutes

Start the timer. Your results will be ready in 15 minutes.


Independently-sealed packaging ensures safe and hygienic testing.

The swabs and test cards of iHealth COVID-19 Antigen Rapid Test are sealed and independently packaged to ensure the safety and hygiene of the product.

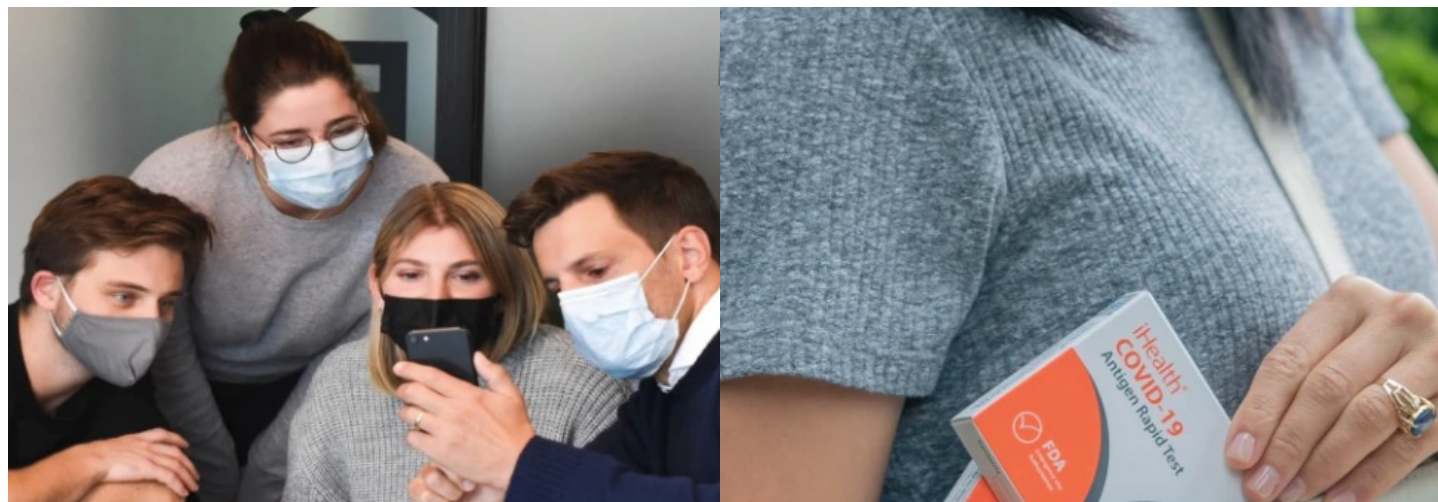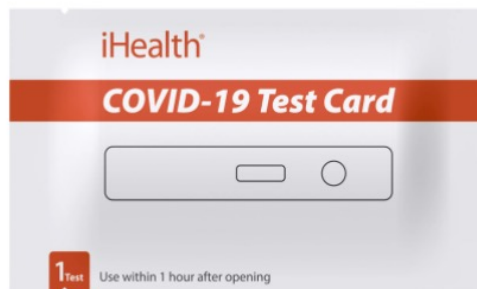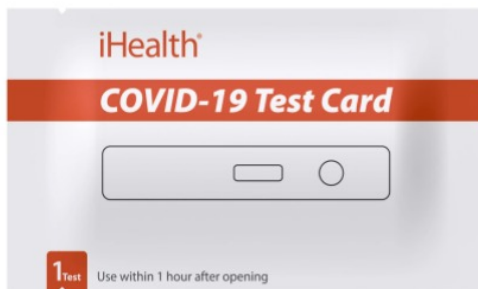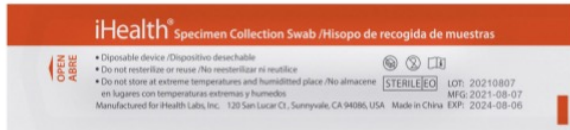

Group COVID-19 testing for schools, event planners, administrators, and organizers everywhere.

Test and monitor results for large groups of people using iHealth in-home testing and the iHealth COVID-19 Test app.

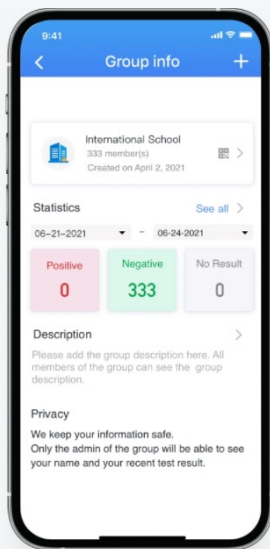


Results Management

Each individual added to the group can test themselves at home and upload their results confidentially to the group manager within a selected timeframe.



iHealth Pass

Test results, once entered in the app, generate an iHealth Pass that can be used as digital proof of negative results for private events.

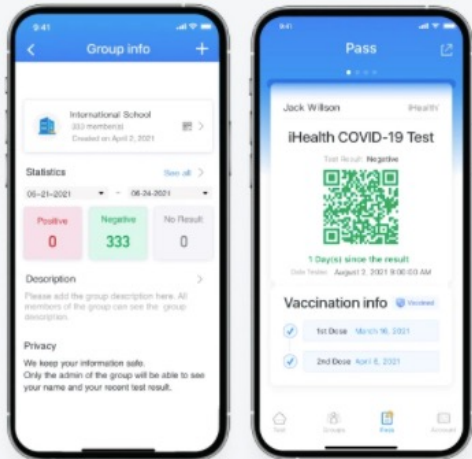

Download the App (Use of App is optional)

Available on the App Store and Google Play.



Scan the QR code
to download the
App

iHealth COVID-19 Antigen Rapid Test

Device Compatibility

Apple iOS devices (requires iOS version 12.0 or higher)

iPhone 7/7 Plus
iPhone 8 Plus
iPhone 11/11 Pro
iPhone 12/12 Pro/12 Pro Max
iPhone SE/ SE 2
iPhone X/XR/XS/XS Max

Android devices (requires operating system 6.0 or later)

LG Stylo 5X
Moto G8 Power
Moto G7 Plus
Samsung Galaxy Note8/Note9
Samsung Galaxy S6/S8/S9/S10/S20
Google Pixel 4a
Huawei P20

Shipping Information

iHealth COVID-19 Self-Test at Home (OTC)

UPC # 856362005890

SKU # COV-AG-2-CTN

Model# ICO-3000

90 Packs per Carton, 2 Tests per Pack

1 pallet: 84 cartons / 7560 boxes

1100 lbs Per pallet

Single Package Dimensions (LxWxH)

6.18 × 3.19 × 0.71 Inches

Single Package Weight (lb)

0.13


Carton Dimensions (LxWxH)

13.1 x 11.8 x 10.8 Inches

Carton Weight (lb)

12.8

Get instant COVID test
results in 15 minutes.

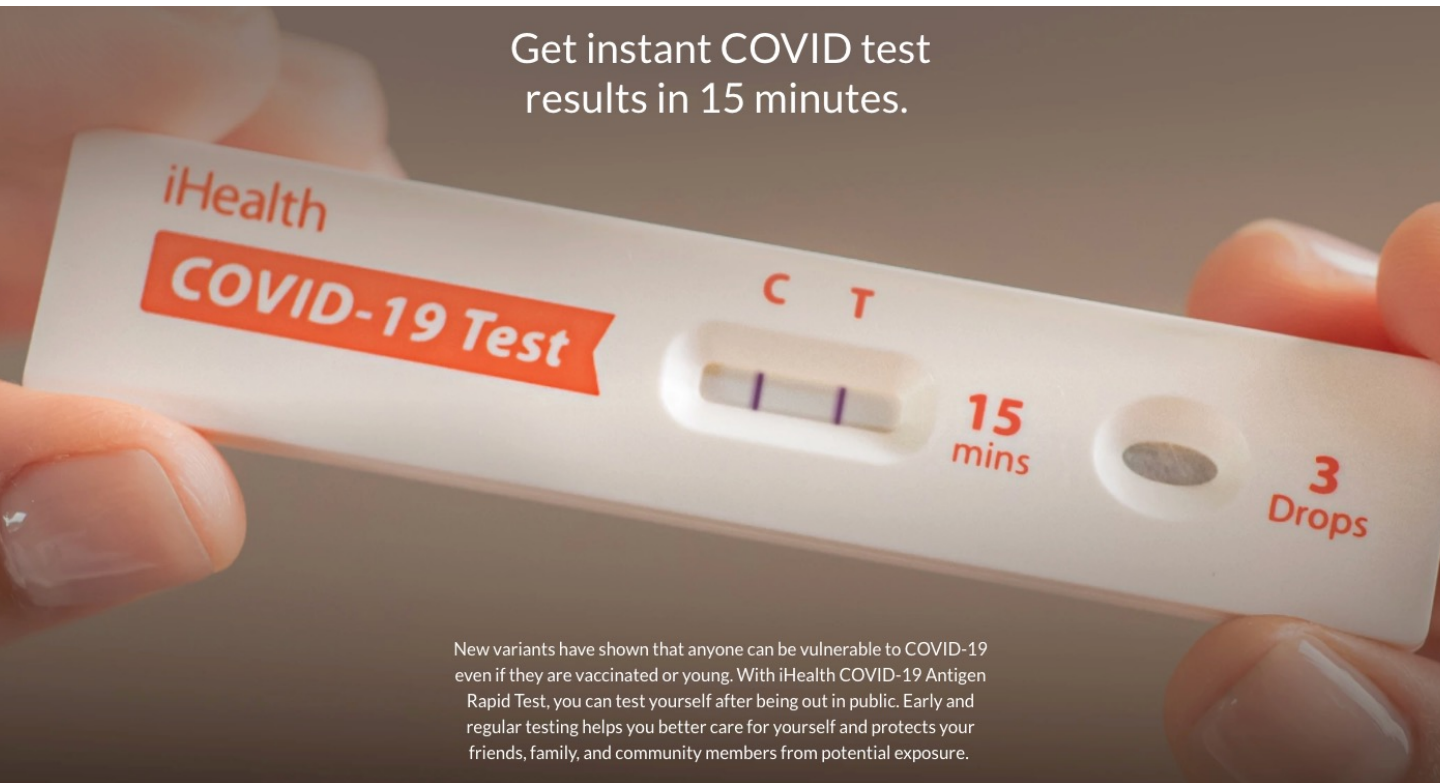

New variants have shown that anyone can be vulnerable to COVID-19 even if they are vaccinated or young. With iHealth COVID-19 Antigen Rapid Test, you can test yourself after being out in public. Early and regular testing helps you better care for yourself and protects your friends, family, and community members from potential exposure.

Product Description

The iHealth® COVID-19 Antigen Rapid Test is a lateral flow assay intended for the qualitative detection of nucleocapsid protein antigen from SARS-CoV-2.

This test is authorized for non-prescription home use with self-collected anterior nasal (nares) swab samples from individuals aged 15 years or older with symptoms of COVID-19 within the first 7 days of symptom onset. This test is also authorized for non-prescription home use with adult-collected nasal swab samples from individuals aged 2 years or older with symptoms of COVID-19 within the first 7 days of symptom onset.

This test is also authorized for non-prescription home use with self-collected anterior nasal (nares) swab samples from individuals aged 15 years or older, or adult collected anterior nasal swab samples from individuals aged 2 years or older, with or without symptoms or other epidemiological reasons to suspect COVID-19 when tested twice over three days with at least 24 hours (and no more than 48 hours) between tests.

